



Your **Flexible Partial Denture** was custom-made for beauty, function, and comfort by expert technicians at Artistic Dental Laboratories.

Visit www.artisticdl.com/resources for care tips.

Proper care is essential. Follow these simple steps to keep your denture in top shape.

ARTISTICDL.COM | 630.679.8686
470 Woodcreek Drive, Bolingbrook, IL 60440

Patient Instructions

Caring For Your Flexible Partial Denture

01

Regularly rinse your flexible denture under running water after meals. Use a sonic denture cleaner for a deeper clean.

02

Avoid alcohol, abrasive cleaners, bleach, whiteners, Listerine, and acidic foods—they can damage or discolor your denture.

03

Always remove your denture before brushing or using mouthwash. Avoid brushing the denture itself to prevent surface damage.

04

Keep your denture in water when not in use to prevent it from drying out.

05

Soak it in denture cleaner for 10-15 minutes daily, or at least three times a week if there's buildup.

06

Never use boiling water, the dishwasher, or the microwave. They can warp your denture.

07

Keep up with your dental appointments, and bring your denture for regular checkups.